Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran
(Copran Zri, CopraSupreme, CopraSupreme Hyperion, CopraSmile)

Basis UDI-DI: ++EWPDCZRMV

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Copran
(Copran Zri, CopraSupreme, CopraSupreme Hyperion, CopraSmile)

Basis UDI-DI: ++EWPDCZRMV

Manufacturer	Whitepeaks Dental Solutions GmbH Alfredstr. 81 - 45130 Essen - Germany SRN: DE-MF-000021120
Product/ Product type	Pre-sintered zirconia blanks for the fabrication of individual dental restorations
Product form	Discs and blocks of various sizes, colours and translucencies, Partly with holder
Material type	ZrO ₂ (yttria-stabilised tetragonal zirconia) / ceramic type 2, Class 5 (exception: CopraSmile products ceramic type 2, class 4) Class IIa medical device
User group	Trained users who individually fabricate dental restorations from zirconium dioxide. produce.

Indication/ purpose

The products of the product group Copran are medical devices intended for the manufacture of dental prostheses for long-term use, which are partially introduced into the human body by means of a clinical intervention, but in this case are inserted into the teeth or attached to the tooth structure with suitable luting material and are thus classified as class IIa.

Copran Zri, CopraSupreme, CopraSupreme Hyperion:

Crowns and bridges up to 16 units with max. 2 pontics between 2 crowns in the posterior region, with max. 4 pontics between 2 crowns in the anterior region, veneers, inlays, onlays, primary telescopes, bar constructions

CopraSmile:

Crowns and bridges up to 3 units in the anterior or posterior region, veneers, inlays, onlays, primary telescopes, bar constructions

Valid for all products from the product group Copran:

The products of the Copran product group are used to mill basic frameworks for crowns, bridges, etc., as well as fully anatomical restorations for dental technology. The restorations can be coloured with the respective liquids of the Copran Color product group, veneered with ceramic and/or glazed.

Contraindication

Do not use in case of proven hypersensitivity to one or more ingredients.
Do not use if there is insufficient space.

Veneering ceramics/ glaze

All ZrO₂ veneering ceramics/ glazes

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran
(Copran Zri, CopraSupreme, CopraSupreme Hyperion, CopraSmile)

Basis UDI-DI: ++EWPDCZRMV

Material properties/ Technical data (values after sintering)

	Copran Zri	Copran Zri Light, Medium, Intense	Copra Supreme	Copra Supreme A1 - D4 Copra Supreme Bleach 0M1 - 0M3 Copra Supreme Symphony A1 - D4 Copra Supreme Symphony Bleach 0M1 - 0M3 Copra Supreme Symphony Joy Five - Nine	Copra Supreme Hyperion A1 - D4 Copra Supreme Hyperion Bleach 0M1 - 0M3 Copra Supreme Hyperion Joy Five - Nine	CopraSmile (up to 3 links in the anterior or posterior region)	CopraSmile A1 - D4 CopraSmile Bleach 0M1 - 0M3 CopraSmile Symphony A1 - D4 (up to 3 links in the anterior or posterior region)
ZrO₂	Balance	Balance	Balance	Balance	Balance	Balance	Balance
Y O₂₃	4,95 - 5,35 %	4,95 - 5,35 %	6,93 - 6,97 %	6,413 - 6,914 %	6,413 - 9,155 %	9,32 %	8,358 % - 9,155 %
Al O₂₃	0,15 - 0,35 %	0,15 - 0,35 %	0,04 - 0,06 %	0,038 - 0,062 %	0,038 - 0,062 %	0,049 %	0,046 % - 0,054 %
Fe O₂₃	0 - 0,01 %	0,04 - 0,25 %	0 - 0,01 %	0,010 - 0,151 %	0,010 - 0,151 %	0 - 0,002 %	0,015 % - 0,142 %
ER O₃₃	0 %	0 %	--	0 - 0,564 %	0 - 0,626 %	0 %	0 - 0,626 %
CO O₃₄	0 %	0 %	--	0 - 0,008 %	0 - 0,009 %	0 %	0 - 0,009 %
Other Oxides	0 - 0,06 %	0 - 0,06 %	0 - 0,02 %	0 - 0,020 %	0 - 0,020 %	0 - 0,002 %	0 - 0,004 %
Density g/cm³	6,05	6,05	6,07	6,07 - 6,33	6,046 - 6,33	6,046	6,046 - 6,33
Bending strength	~1400 MPa	~1400 MPa	~1100 MPa	~1100 MPa	~600 Mpa (cutting area) to ~1100 Mpa (Cervical / Dentine range) *	~600 MPa	~600
WAK (25°C -500°C)	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶	10.5 +/- 0.5 x 10 /K ⁻⁶

*The placement of the restoration within the blank must be observed. See processing

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

Copran (Copran Zri, CopraSupreme, CopraSupreme Hyperion, CopraSmile)

Basis UDI-DI: ++EWPDCZRMV

Processing

The desired restoration is milled from the selected blank:

The wall thickness for anterior teeth must not be less than 0.5mm and for posterior teeth 0.6mm. The junction between two splinted anterior crowns must not be less than 8mm². The connection between an anterior crown and a pontic, two splinted posterior crowns or two anterior pontics must not be less than 9mm². The connection between a posterior crown and a pontic or two posterior pontics must not be less than 12mm². The cross-section of a bar construction must not be less than 15mm .²

With CopraSupreme Hyperion, the neck/dentin portion always accounts for 50% of the blank height. The connection point must be placed in this area, i.e. always in the lower half of the blank, to ensure sufficient flexural strength.

The dimensions refer to the dimensions after sintering for all product variants.

The enlargement or shrinkage factor is indicated on the blank depending on the milling system used.

The unsintered restorations are still finished if necessary. In order to obtain a perfect surface and maximum translucency, we recommend cleaning the restoration in the ultrasonic cleaning unit "White-Sonic" with distilled water. Please use only our recommended ultrasonic cleaning device. Devices from other companies could be too "strong" and damage the restoration. Swivel the restoration with plastic tweezers in the distilled water of the ultrasonic unit for 5 to 10 seconds until no more "dust cloud" comes off the restoration. Remove excess water from the restoration by blowing it off with oil-free air or by blotting it dry with cellulose or cotton swabs. Dry the restoration under an infrared lamp or using an oven. Before sintering, the restoration must be completely dry. Make sure that the water never boils during drying, as this may cause cracks in the zirconia.

If colouring is desired, this can be done with the corresponding liquids from the Copran Color Group. Please also refer to the processing instructions.

The finished, completely dry restoration is now sintered.

Sintering

	Normal Program	Slow Program	Translucency Program	Speed Program
Heating rate	10°C per minute to 950°C	5°C per minute to 950°C	5°C per minute to 950°C	50°C per minute to 1100°C
Holding time	none	none	none	none
Heating rate	6°C per minute to 1500°C	2°C per minute to 1500°C	2°C per minute to final temperature	20°C per minute to 1500°C
Holding time	at final temperature 90 minutes	at final temperature 120 minutes	at final temperature 120 minutes	at final temperature 30 minutes
Final temperature	1500°C	1500°C	1500°C - 1630°C	1500°C
Cooling down	unregulated in the closed oven	unregulated in the closed oven	unregulated in the closed oven	unregulated in the closed oven

Post-processing

After sintering, the restoration is adjusted to the working model, if necessary, by wet grinding with diamond-coated grinding instruments. Sintered diamonds, corundum stones or carbide burs should not be used. Overheating must be avoided.

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

**Copran
(Copran Zri, CopraSupreme, CopraSupreme Hyperion, CopraSmile)**

Basis UDI-DI: ++EWPDCZRMV

Firing the ceramic/glaze

All commercially available firing-on ceramics/glazes for zirconium dioxide can be used. Please observe the working instructions of the respective manufacturer.

Firing and cooling according to the firing chart of the manufacturer of the ceramic mass/glaze used.

Safety instructions

Warning: Always wear respiratory protection (filter class FFP2), tight-fitting safety goggles, protective gloves and clothing and switch on extraction equipment. The dust generated by this product during processing may cause irritation to the skin/eyes/respiratory tract. Avoid inhalation/ contact with skin, mouth and eyes. Do not eat or drink while working. Keep away from food and drink. Wash hands after use. Remove contaminated clothing and protective equipment before entering areas where food will be eaten. Keep away from sources of ignition. Do not smoke .

Storage

Store in a dry place. Protect from moisture. Store in the original packaging.

Disposal

Dispose of product and packaging according to local/ regional/ national/ international regulations. Do not dispose of together with household waste. Do not allow to enter waters, groundwater or sewage system.

Note

All serious incidents related to the device shall be reported to the manufacturer and to the competent authority of the Member State where the user and/or the patient is established.

The summary report on safety and clinical performance according to (EU) MDR 2017 / 745 Article 32 (SSCP) is available in the European Database for Medicinal Products (Eudamed) under the associated base UDI-DI as soon as the corresponding database module exists.

<https://ec.europa.eu/tools/eudamed>

Explanation of the markings on the packaging

Symbol for "Article number"

Symbol for "batch designation"

Confirmation: The product complies with the applicable European directives

Symbol for "Number of products in the package".

Symbol for "Follow instructions for use"

Symbol for "is a medical device"

Symbol for "Date of manufacture"

Rx only

Symbol for "Caution: US Federal law restricts this device to sale by or on the order of a licensed physician or dentist."

Gebrauchsanweisung/ Instructions for use/ Manuel d'instruction/ Manuale d'istruzioni/ Manual de instrucciones/ Instrukcja obsługi/ Bruksanvisning/ Instruksjoner for bruk/ Käyttöohjeet/ Gebruiksaanwijzing/ Εγχειρίδιο οδηγιών/ Upute za korištenje/ Manual de instrucțiuni/ Instrukcija/ Návod k použití/ Návod na použitie/ Használati utasítás

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