



IMPORT knižníc pre 3Shape verzia 2022:

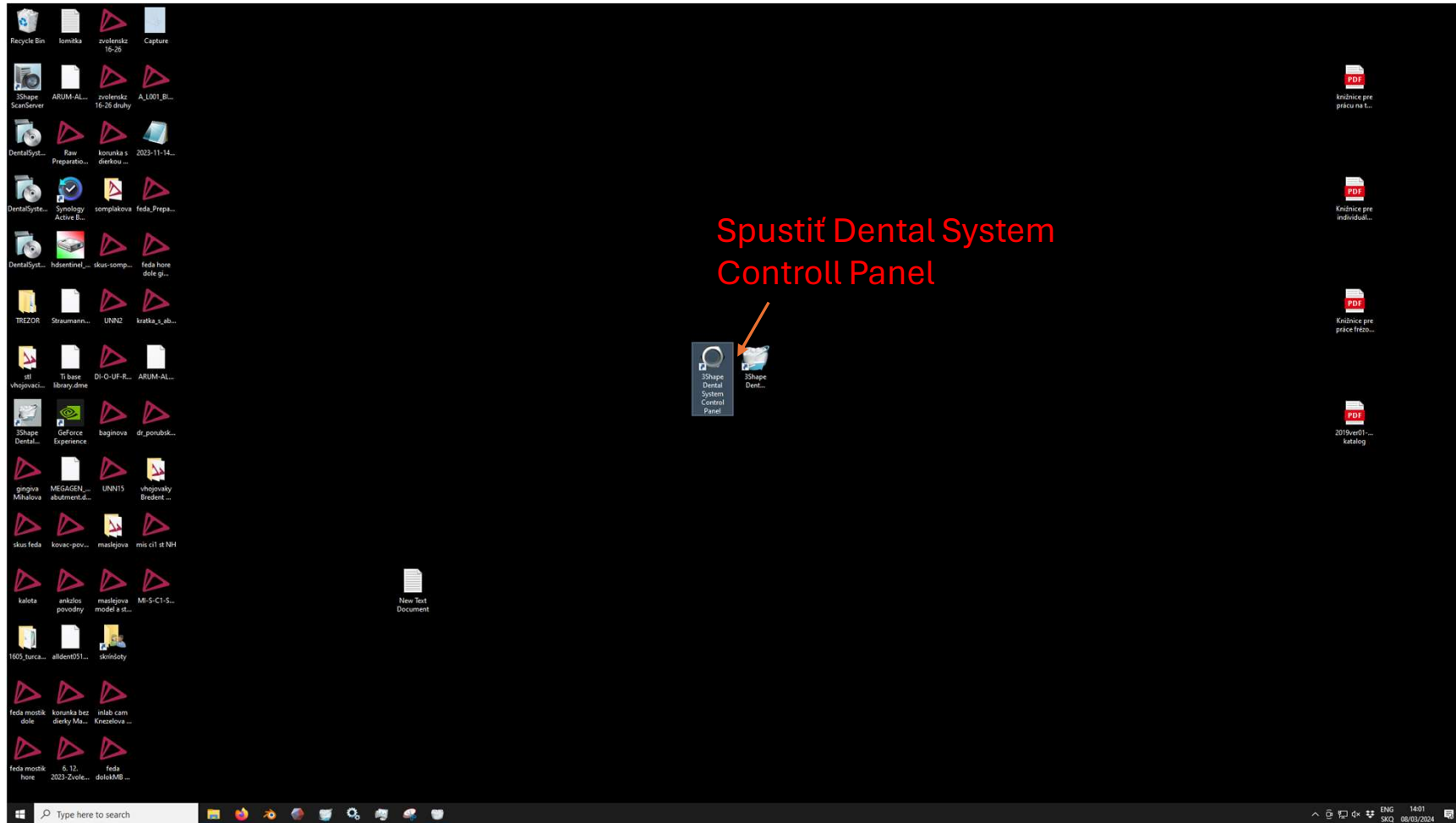


IMPORT knižníc pre 3Shape verzia 2022:

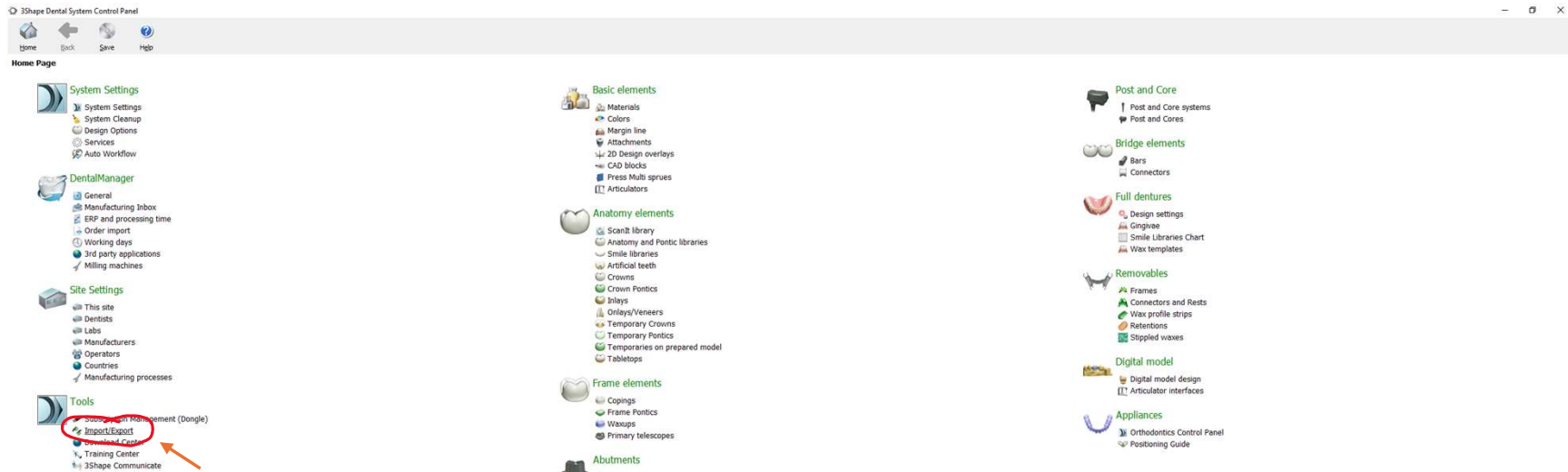

Kliknúť na Import/Export

IMPORT knižníc pre 3Shape verzia 2022:

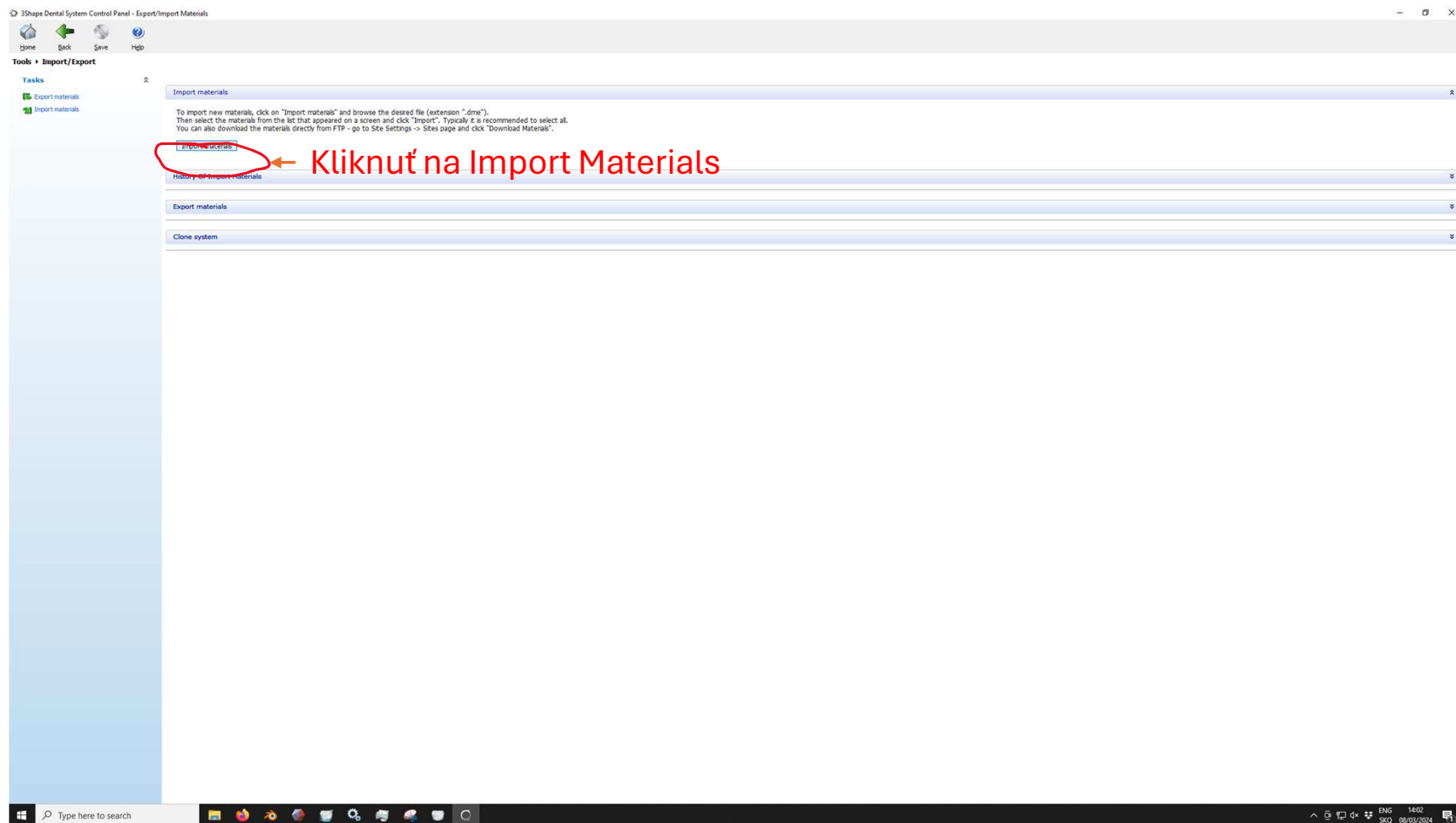

IMPORT knižníc pre 3Shape verzia 2022:

The screenshot shows the 3Shape Dental System Control Panel interface. The 'Import materials' window is open, displaying instructions for importing materials. A file explorer window is overlaid on top, showing a directory structure. The file 'ARUM 3D Dental Solutions Premill and Disk.dme' is highlighted with a red circle. An arrow points to this file with the text 'Nájsť knižnicu ARUM ktorú ste si stiahli a dvojkliknúť na prvú knižnicu'.

Name	Date modified	Type	Size
ARUM 3D Dental Solutions Premill and Disk.dme	02/03/2024 04:33	DME File	20,917 KB
ARUM 3D Dental Solutions - Premill		DME File	14,493 KB
Arum Locator Male direct mill.dme		DME File	24 KB
Arum Model-Raw STL-Zircon.dme	02/03/2024 04:33	DME File	15 KB

Nájsť knižnicu ARUM ktorú ste si stiahli a dvojkliknúť na prvú knižnicu

IMPORT knižnic pre 3Shape verzia 2022:

3Shape Dental System Control Panel - Export/Import Materials

Tools > Import/Export

Tasks

- Export materials
- Import materials

Import materials

To import new materials, click on "Import materials" and browse the desired file (extension ".dme"). Then select the materials from the list that appeared on a screen and click "Import". Typically it is recommended to select all. You can also download the materials directly from FTP - go to Site Settings -> Sites page and click "Download Materials".

[Import materials](#)

History Of Import Materials

Export materials

Clone system

Import and update materials

Please select materials to import

- Available materials
- Implant systems
- Materials

Library name

Provider

Library type

General library

Version

Library creation date

29/02/2024

Description


Import Cancel

Označiť všetky 3 možnosti a potom kliknuť na Import

Type here to search

ENG SKQ 14:04 08/03/2024

IMPORT knižníc pre 3Shape verzia 2022:

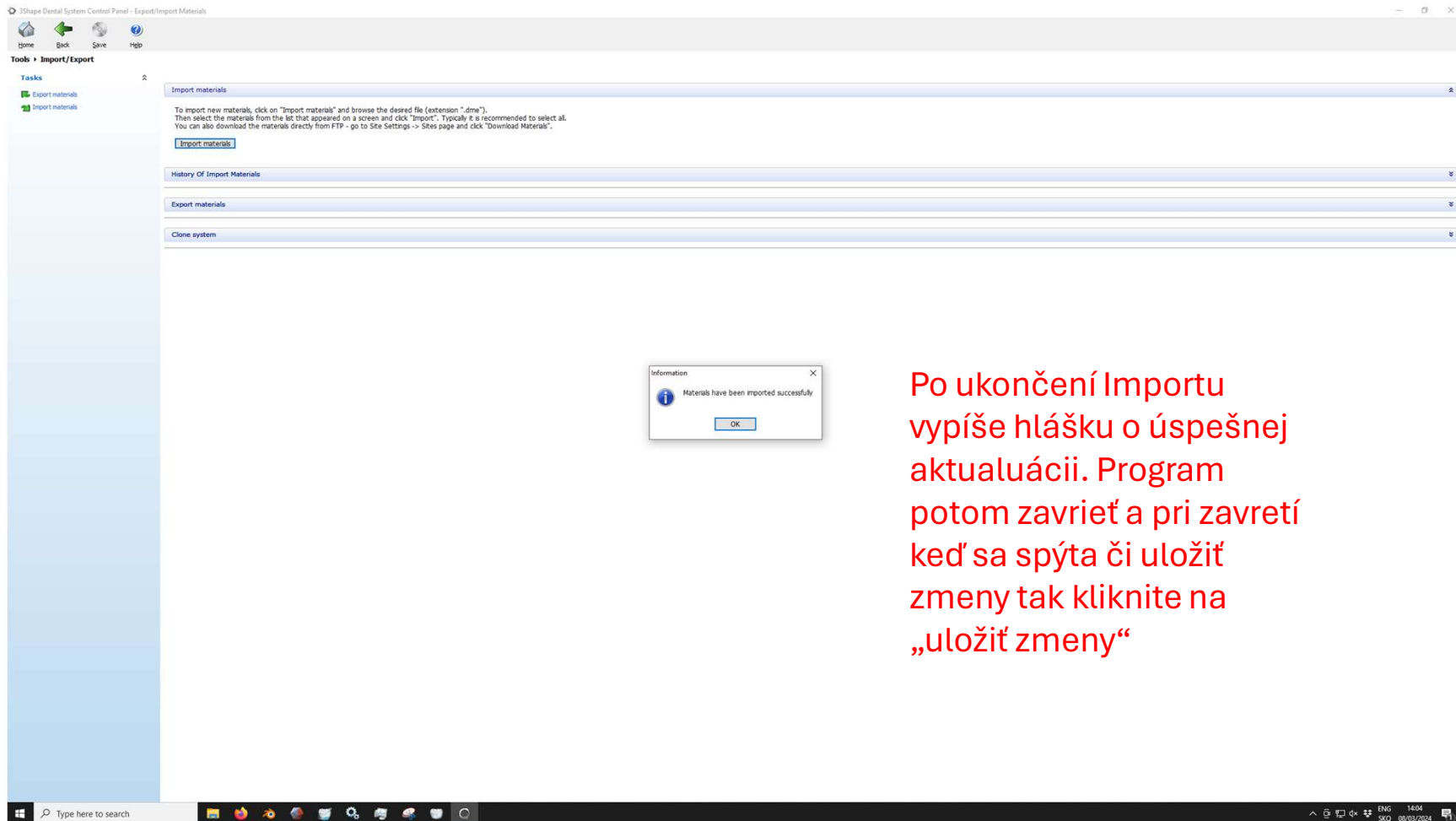

Po ukončení Importu vypíše hlášku o úspešnej aktualizácii. Program potom zavrieť a pri zavretí keď sa spýta či uložiť zmeny tak kliknite na „uložiť zmeny“